

● **What is NFC?**

**NFC** (Near Field Communication) enables QuickLink™ download of your blood glucose results from your GlucoMen Areo meter to your smartphone or tablet, simply by holding them back-to-back!

● **FREE GlucoMen Apps**

QuickLink™ download is easy and fast!

Install a dedicated GlucoMen app onto your mobile device and you will be able to:



Instantly view your logbook and graphs with the GlucoLog® Lite app and send them by email to your healthcare professional.



Send and share your results in real time by text message with the **GlucoLog® Mobile** app.

**Initial preparation**

**Step 1**

**DOWNLOAD**



**Step 2**



- Download and install the **FREE** GlucoMen app; **GlucoLog® Lite** and/or **GlucoLog® Mobile** on to your smartphone and/or tablet device.

- Enable the **NFC** function on your **Smartphone / Tablet** device.

- Enable the **NFC** function on your **Glucomen® Areo** meter following these simple steps:
  - 1) Press **⏻** for 4 seconds to enter setup menu.
  - 2) Press **▲** or **▼** to scroll through the settings; once you reach **N**) press **⏻** to select.
  - 3) Press **▲** or **▼** to select **ON**, then press **⏻** to confirm and exit.

**Download data**



- Hold the **Glucomen® Areo** meter and your smartphone/tablet device **BACK to BACK** for 1-2 seconds.
- The data transmission will happen automatically and when complete, the meter will display **'END'**.

The brands indicated belong to the legitimate owners.