



1 Wash and dry hands thoroughly.



2 Insert a test strip fully into the meter.



3 Insert the lancet and remove finger pricker cap.



4 Set lancet depth.

GlucoMen[®]LX²

GUIDE TO BLOOD GLUCOSE TESTING



5 Prime the finger pricker.



6 Prick your finger (Try to avoid the pads of your fingertips).



7 Apply the blood to the end of the test strip.



8 The result is displayed in just 4 seconds.

GlucoMen[®]LX²

QUICK START GUIDE

- Update your prescription
- Register your GlucoMen LX 2 meter
- How to perform a blood glucose test



Follow GlucoMen on Social Media



www.glucomen.co.uk

PRESCRIPTION UPDATE REQUEST

Dear Doctor

I have upgraded my blood glucose meter to the **GlucoMen® LX 2** and will require the following ticked items as part of my repeat prescription:

- | | |
|---|--------------------------|
| <input type="checkbox"/> GlucoMen LX Glucose Sensors (50) | PIP Code 332-8226 |
| <input type="checkbox"/> GlucoMen LX Ketone Sensors (10)
(Only tick if you have been told to test for blood ketones) | PIP Code 358-2566 |
| <input type="checkbox"/> Glucoject Lancets PLUS 33G (100) | PIP Code 385-1748 |
| <input type="checkbox"/> Glucoject Lancets PLUS 33G (200) | PIP Code 385-1755 |

Name _____ Signature _____

If you have any queries please contact **GlucoMen** on:

GB: 0800 243667 N. Ireland: 0800 7837286 ROI: 1800 709903

Email: glucomen@menarinidiag.co.uk



405 Wharfedale Road, Winnersh, Wokingham, Berkshire, RG41 5RA

Register your **GlucoMen® LX 2** meter to activate your lifetime product warranty. You can order free replacement batteries and logbooks either online or by calling our Customer Support Team, Mon-Fri 8:30am-5:30pm.

FREEPHONE: 0800 243667 or **0118 9444128**

ONLINE: www.glucomen.co.uk/register

Republic of Ireland & Northern Ireland Residents

TELEPHONE: Republic of Ireland **1800 709903**

Northern Ireland **0800 7837286**

ONLINE: www.mediconire.com